



XpertEye Essential - v4.2

Release Note

29/04/2019



Hello,

We are pleased to announce the release of Xperteye Essential version v4.2.

TABLE OF CONTENT

Introduction	3
General functionalities	3
Collaboration	4



INTRODUCTION

You will find in this document the list of features included in the new version of **XpertEye Essential** product.

GENERAL FUNCTIONALITIES

RealWear glasses work with voices commands.



As an XpertEye Essential user, I can:

- ✓ Configure Wi-Fi connection and authentify to XpertEye by generating a QR code :

1. To generate QR code, please connect to the website : <https://config.xpertheye-live.com>
2. On RealWear glasses :



To configure Wi-Fi, say « CONFIGURATION », then scan the associated QR code.



To authentify to XpertEye, say « XPERTEYE », then scan the associated QR code.

- ✓ Be called and participate to a conference :



Say “Accepted” or Say “Declined”


- ✓ Activate or deactivate microphone and camera during a call :



1. Say “Commands”
2. Say “Camera” and/or Say “Microphone”

- ✓ Visualize the board during a call

- ✓ Configure the interface language

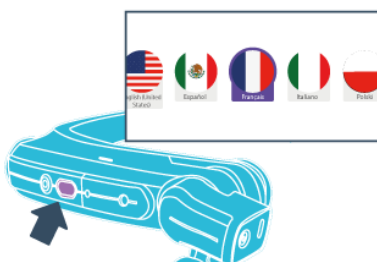

Press and hold the action button until you reach the desired language. When the desired language is highlighted, release the Action button.

Note: The XpertEye application is available in French and English.



As an Xperteye light user, I can:

- ✓ Call a RealWear Glasses wearer Essential version and add a RealWear Glasses wearer Essential version to a conference



Call in One-to-One mode



Create a conference

- ✓ Save the chat content, notes and report at the end of a call
- ✓ Detect the browser by default

If a browser other than Chrome is detected, this functionality will offer Chrome download.

<XPEREYE>

We have detected that you are using Firefox Web Browser. To use XpertEye, please use the latest version of Chrome.



COLLABORATION

You have the possibility:

- ✓ For distant participant, to take picture during a call

By clicking on the button



- ✓ For distant participant, to launch board and use collaborative functionalities of XpertEye photoboard

By clicking on the button



- ✓ For distant participant, to open PDF file on the board
- ✓ For distant participant, to use pointer, during a call or a conference, visible by all participants

CONTACTS

TECHNICAL SUPPORT



User Manual, technical documentation, video tutorial, help...

Visit our Support website: <https://support.amaxperteye.com>



Phone : **(+33) 2 55 59 09 22**

Email : support.emea@amaxperteye.com



Phone : **(+1) 857 271 5001**

Email : support.northamerica@amaxperteye.com



COMMERCIAL TEAM

Europe

: **+33 2 23 44 13 39** - : **+44 330 6845077**

: **+49 221 55405070** - : **+40 728 131 815**

America

: **+1 617 440 7672**