TABLE OF CONTENT

Instructions for Use

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Epione Platform 1.x.x

Diffusion date: 2020-10-14

Instructions for Use

Language version: English

Epione Platform 1.x.x – 2020-10-14
Instructions for Use – Language version: English

Instructions for use are supplied electronically, paper format is available on request at support@epione.care

1. Applicable symbols

Symbol	Detail
	European conformity mark
	Electronic instructions for use are available in the Epione Platform by clicking on this symbol
	Epione BV Groeneweg 35A 3090 Overijse Belgium
	This symbol is to indicate an important warning in these Instructions for Use. This symbol can vary in size.

2. Product description

Epione is a software platform on advice, intended for patients and healthcare professionals to monitor the patient’s chronic neurological disorders. The Epione Platform consists of two Epione Applications and an Epione Dashboard. Together with the Epione Dashboard, each Epione Application is specific for a particular chronic neurological disorder:

- The Epione Application Helpilepsy and the Epione Dashboard are specifically intended for patients and healthcare professionals to monitor the patients’ epilepsy
- The Epione Application MigraineManager and the Epione Dashboard are specifically intended for patients and healthcare professionals to monitor the patients’ migraine.

GENERAL DESCRIPTION

Once the user is registered on an Epione Application, on advice by his healthcare professional, he can enter manually or optionally through the use of a connected device, information about his disease, health and medicines. Inputs from the user to the application relate to the following:

- Recording attacks, side effects and health-related behaviors
- Daily rating of mood and sleep
- Recording treatments and medication intake
- Planning appointments with the healthcare professional
- Response to the questionnaire sent by his healthcare professional
- Response to questionnaires that are sent in the context of a clinical study

The different outputs of the application are:

- Visualization of the entered data
- Generation of a report based on the entered data
- Notifications on the medication intake and planned appointments

The healthcare professional gets access to the generated report and the user's data through the Epione Dashboard. The report and Epione Dashboard are used to analyze the patient's data and therefore, to support the medical appointment and the decisions related to the medication intake. As the Epione Applications interact with the Epione Dashboard, the inputs relate to the user and to the Epione Applications as follows:

- Patient's data received from the Epione Applications
- Questionnaires the healthcare professional would like to see answered by the patient
- Patient's data added in the context of a clinical study

The outputs of the Epione Dashboard are:

- Comparison of patient's treatments on attacks and side effects
- Timeline of attacks with type and count and other health-related events
- List of reported attacks
- Summaries between appointments
- Overview of data that is gathered in the context of a clinical study

To help with the communication, the Epione Dashboard also comprises a video-consultation module that allows the healthcare professional and the patient to hold medical appointments remotely.

INTENDED USE

People suffering from chronic neurological disorders, such as epilepsy or migraine, can use the Epione Applications on a daily basis to record their symptoms, to better track and have a better understanding of their disease. Recordings are entered on the smartphone and are automatically shared with the healthcare professional using the internet. Healthcare professionals can use the Epione Dashboard to analyze the patient's data and to monitor and manage the patients which are using the Epione Applications.

The Epione Platform, consisting of the Epione Applications Helpilepsy & MigraineManager and Epione Dashboard, is a Class I medical software intended for patients and healthcare professionals to monitor the patients' chronic neurological disorders.

Note: this medical device is no substitution for normal care or practice. The data displayed is informative but may be useful to the doctor to support treatment decision. A doctor should always be contacted in case of any symptoms that are not within the expectations.

USERS

The Epione Applications Helpilepsy and MigraineManager are used by the patients and their relatives, while the Epione Dashboard is used by the patients' healthcare professional.

3. Important safety information

GENERAL SAFETY INFORMATION FOR ALL USERS

To use the Epione Platform, each user should:

- Have an email address
- Have basic computer and internet knowledge
- Be willing to use the Epione Platform

GENERAL SAFETY INFORMATION FOR EPIONE APPLICATIONS USERS

When using the Epione Applications Helpilepsy and MigraineManager, keep these points in mind:

- It is recommended to set the settings in the App Store and Play Store so that the Application is automatically updated when a new update becomes available. If not, please update to the newer version as soon as possible.
- The Application is not intended to replace the physician, so in case of any question or emergency, please consult your treating physician.
- We recommend filling in daily information about your health (e.g. attacks, symptoms, medication, sleep quality) and use it for a minimum of 1 year with your treating neurologist.

GENERAL SAFETY INFORMATION FOR EPIONE DASHBOARD USERS

When using the Epione Dashboard, keep these points in mind:

- When inviting patients, please ensure that they fill the usage requirements (see "Users" in "1. Product Description").

PERFORMANCE ISSUES

If you experience error messages or slow loadings, verify that your internet connection is stable. If the problem persists, contact the Epione team via support@epione.care and stop relying on the device till the Epione team resolved the issue.

DATA PROTECTION & DATA SECURITY

Epione uses industry-standard encryption protocols to secure all products and data in transfer and at rest. The data is only stored on secure, encrypted servers and not on the device of the user. A continuous backup system is used to make sure all data is secure. Other technical and organizational measures exist within Epione to keep the data safe and accessible.

You, as a user, can also improve the safety of your data, or your patients' data. For this, it is very important to keep in mind the following recommendations:

- Use a complex and unique password for your login:
 - o For Epione Applications users, we recommend minimum 8 characters, with at least 1 digit, 1 capital letter and 1 special character;
 - o For Epione Dashboards users, you are requested to use a password with minimum 8 characters, with at least 1 digit, 1 capital letter and 1 special character;
- If you use the Epione Dashboard, we recommend setting up 2-factor-authentication. This can be done in the “My settings”-page of the dashboard;
- Be careful to never share your credentials;
- When using a smartphone from someone else for the Epione Applications, don't forget to log out of them;
- When using a public computer, don't forget to log out of your Epione session;
- Use a secured Internet connection

Furthermore, as a user, you have several rights with regards to your data. These rights are explained in our [Privacy Notice](#). For all questions and concerns regarding the security of your data, please contact us at privacy@epione.care.

VERIFYING CORRECT INSTALLATION

Epione Dashboard

To verify you are using the latest version of the Epione Dashboard, on the home page of the Epione Dashboard, you can check the version number, which should fit with the version reported in these Instructions for Use.

Epione Applications Helpilepsy & MigraineManager

To verify you are using the latest version of an Epione Application (Helpilepsy or MigraineManager), go to the App Store from Apple or the Play Store from Google and check the application if an update is available. If up to date, you can see in the home page (or in the Settings-page of the app) the version number, which should fit with the version reported in these Instructions for Use.

CONNECTED DEVICES

The Epione Platform can be connected to third party devices.

For now, only the NightWatch from LivAssured, a medical device under European regulation, can be connected. Please note that we are not responsible for this device. For more information about this device, please go to <https://www.nightwatchepilepsy.com>.

4. Steps for use

EPIONE DASHBOARD

In this chapter, you can read the guidelines on how to use the Epione Dashboard, intended for healthcare professionals.

Setting up your physician account

You need a personal account to monitor and manage the health of your patients. Go to the Epione Dashboard and use the “Create new account”-form.

Important: To create a “healthcare professional” account, you need to have received an Activation Code. To request one, contact us at support@epione.care.

After this, follow the instructions presented.

Inviting a patient to use the Epione Applications

To invite a patient, go in the “My patients”-page and click on the “Add a new patient”-button. Fill in the required and optional fields. Once the patient is added, he will receive an invitation email to start using the Epione Application Helpilepsy or MigraineManager and will be prompted to accept sharing his data with you.

If the patient hasn't created an account yet or has denied the data sharing, you can prompt again your patient to do so by clicking “Resend invite” or “Ask to share data” in the list of patients in the “My patients”-page.

Monitoring the data of your patient

In the “My patients”-page, tap on a patient in the list of patients to go to that patient's dashboard. In there, you have access to several views:

- Treatments: This page shows the comparison between different treatments and the effect on the patient.

- Timeline: This page shows the data of the patient in a timeline view, in different frequencies.
- Seizures / Headaches: This page shows the logs of seizures / headaches logged by the patient.
- Appointments: This page shows a summary of information between appointments.
- Medications: This page shows the list of medications used by the patient.
- Forms: This page shows scheduled and answered forms of the patient.
- Videos: This page shows a list of seizure videos of the patient connected to a specific seizure event.
- (optional) Nightwatch: This page shows the logs of the NightWatch device used by the patient.
- (optional) Specific study pages: This page shows the details of the patient in the context of specific clinical study.

Sending a questionnaire / form to your patient

As a healthcare professional, you can send questionnaires / forms to your patient. These can be related to quality of life, comorbidities, side effects, symptoms, appointment preparations, etc.

To send them, go to a patient in the “Forms”-page and click on “Send a new form”.

If you would like to add new questionnaires / forms in the list, contact us at support@epione.care.

Adding a (video) appointment to a patient

As a healthcare professional, you can add appointments with your patient through the Epione Dashboard. To do this on an existing patient, go in “My patients”-list and click on the “+” icon next to the patient. Fill in the form.

If you select a video appointment, your patient will receive an invitation link to join the video appointment. To manage and join your video appointments, click on “My video appointments”-page and join/edit appointments.

Important: The video appointments do not work on the browser Microsoft Internet Explorer. Please use an alternative like Google Chrome, Mozilla Firefox or Microsoft Edge.

To invite a patient not using the Epione Applications yet, you need to wait that he created his account or invite him for a video appointment (in that case, he will receive the link to the video consultation without having to create his account).

If you are using video appointments in the Epione Dashboard, verify the identity of the patient before sharing confidential information.

Participating to a study

Via “My studies”-page you can access the studies to which you are participating. Your own studies can also be configured. For more information, contact us at support@epione.care.

EPIONE APPLICATIONS HELPILEPSY & MIGRAINEMANAGER

In this chapter, you can read the guidelines on how to use the Epione Applications Helpilepsy and MigraineManager.

Setting up your personal account

You need a personal account to monitor and manage your health with the Epione Applications Helpilepsy & MigraineManager. Setting up an account can be done in 2 steps. Follow the step-by-step instructions below to successfully create an account.

i. Downloading the Helpilepsy or MigraineManager application

If your physician invited you to use the Application, you will find a link in the invitation email to download the application on the App Store from Apple or the Play Store from Android. Download the app and open it.

If your physician provided you with his Email Account, go to the App Store from Apple or the Play Store from Android and search “Helpilepsy” or “MigraineManager”, download the app and open it.

ii. Registering your personal account

Go through the short tutorial screens and then tap on the “Create new account”-button. Fill in the registration form.

Important: If you were invited by your physician, please use the same email address on which he invited you! You can find this email address in the invitation email you received.

After this, follow the instructions presented.

An important step is the sharing of your data with your physician that invited you:

- If you were invited by your physician via email, you will be prompted to approve the data sharing with your physician, accept it (after having controlled it is the right physician name displayed).
- If your physician provided you his email address for you to add in the Application, add that email address in the appropriate step during the registration procedure.

Adding health-related events & answering questionnaires

It is recommended to add daily health-related events in the Application. You can do this by tapping the “Add +”-button on top-right of the “Home” and “Diary” screens. You can add several types of events:

- Seizure (Helpilepsy) or Headache episode (MigraineManager)
- Appointments

- Side effects
- Notes/events: General event to add any noteworthy event related to your health. In case of doubt, please add it.
- Daily mood & sleep quality

For each event, several fields are optional, however it is recommended to document for each event as much details as possible to use the full potential of the product.

Depending on your physician, you will be prompted once in a while to answer questionnaires. For this you will receive notifications. Tap on them to answer the questionnaires.

Setting up your medication

It is recommended to report all your medication, not only linked with epilepsy or headaches, in the Application. For this, go to the “Treatments”-page and tap “Add new treatment”. From there, follow the step-by-step instructions.

Take care to record the right information as this will support the medical appointment and the decisions related to medication intake.

Several fields are optional, however it is recommended to document for each medication as much details as possible to use the full potential of the product.

You can also setup or deactivate medication intake reminders: tap on “Edit intake hours & reminders” in the “Treatments”-page.

5. Compatibility

EPIONE DASHBOARD

Software compatibility

The Epione Dashboard, an online tool for the Healthcare Professional, is compatible with:

- Google Chrome (83 or higher)
- Internet Explorer (11 or higher) (! Except the Video Appointments module)
- Mozilla Firefox (76 or higher)
- Apple Safari (13 or higher)
- Microsoft Edge (83 or higher)

Users will always have access to the latest version of the Epione Dashboard since it's a secured website. They will be notified of minor and major updates.

Hardware compatibility

In order to use the Epione Dashboard, the device must be constantly connected to the Internet and meet the following minimum hardware requirements:

- 1.3 GHz CPU
- 1GB RAM
- 8GB Internal Storage

EPIONE APPLICATIONS HELPILEPSY AND MIGRAINE MANAGER

Software compatibility

The minimum system requirements for the Epione Applications Helpilepsy and MigraineManager are an Android operating system with minimal software version Android 4.4 or an iOS operating system with minimal software version iOS 11.0.

Epione users will be notified when an update is available. The update of the Epione Applications can be found in the App Store/ Play Store.

Hardware compatibility

In order to use the Epione Applications, the device must be constantly connected to the Internet and meet the following minimum hardware requirements:

- 1.3 GHz CPU
- 1GB RAM
- 8GB Internal Storage

6. Support

If you have any questions or if you found a problem about the Epione Platform, including the Epione Applications Helpilepsy and MigraineManager, please contact us at support@epione.care.

In the event of a serious incident related to the use of this medical device, you can contact Epione customer service desk (via support@epione.care), as well as the responsible local authority.

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